



position A



position B

P-Bag Usage Directions



Attach Funnel: Funnel should be inserted into the circular portion of the P-Bag by applying a small amount of pressure until it snaps into place.



For Women: Place the funnel right below the urethra and apply a small amount of pressure to ensure a good fit.




For Men: Use the P-Bag as if it is a urine bottle.




Sitting Position: When sitting the handle should be facing the upper side of the P-Bag (See position A)




Vertical Position: When standing, the handle should be facing the upper side of the P-Bag (See position A)

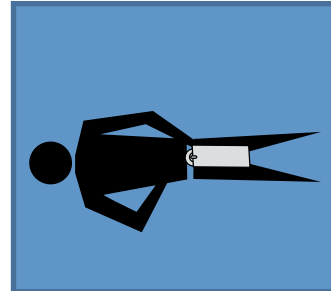


Horizontal Position: When laying down, the handle should be facing the bottom side of the P-Bag (See position B)

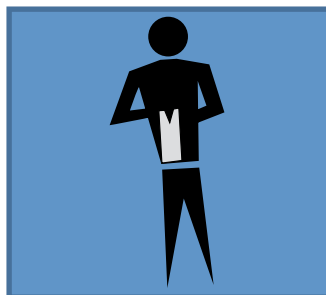

Remove Contents: Tear Bag on V at top of bag pinching with both thumb and pointer fingers.

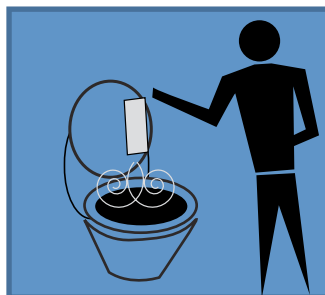

Contents Disposal: Empty contents into toilet.

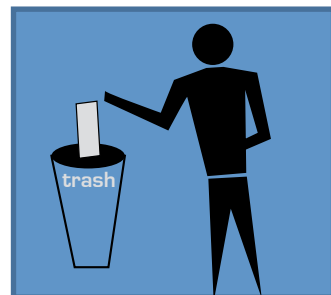

Bag Disposal: P-Bag can be disposed of in a normal manner. Funnel is re-usable.